



Ready to use, aqueous solution for the rapid wiping disinfection of particularly sensitive surfaces and medical devices

Fields of application:

Acrylan® is a ready to use solution for the quick wiping disinfection of surfaces. The product distinguishes itself through its **excellent material compatibility**.

Acrylan® is based on a QAC (quaternary ammonium compound) and is free of alcohols and aldehydes. Therefore, it is ideally suited for the application onto alcohol-sensitive materials like acrylic glass.

Acrylan® offers a broad spectrum of efficacy and good cleaning properties. It is effective within short contact times and has a pleasant, neutral scent.

The product is suitable for the disinfection and cleaning of medical devices and surfaces, e.g. in ICUs, examination tables, stretchers, and protective mattress covers in doctors' practices, accident and emergency units and physiotherapy; also for ambulance surfaces, ultrasound devices, tonometer measuring prisms and contact glasses (e.g. Haag-Streit), etc.

Due to its outstanding material compatibility, Acrylan® is also perfectly suitable for the disinfection and cleaning of surfaces made of acrylic glass and plastic covers in industrial plants/production sites, solarium, etc.

Recommendations for use:

Test Concentration Contact time

VAH – Surface Disinfection with mechanical action | DGHM Standard Methods

VAH – Surface Disinfection: bactericidal, yeasticidal clean and dirty conditions (equivalent to EN 13727, EN 14561, EN 13624, EN 14562)	undiluted	5 min
---	-----------	-------

DVV / RKI Guideline – virucidal efficacy

limited virucidal* (inactivation of enveloped viruses) (incl. HIV / HBV / HCV)	undiluted	5 min
Vacciniavirus	undiluted	5 min
BVDV (Bovine Viral Diarrhea Virus)	undiluted	1 min
Rotavirus	undiluted	5 min

* according to RKI recommendation, Bundesgesundheitsblatt 01/2004

Special characteristics:

- suitable for the use in the wipes dispensing system **A°-Wipes** and **Kimtech® Wettask® DS**
- wide range of application
- fast onset of action
- very high material compatibility even with particularly sensitive materials like acrylic glass
- pleasant scent

Composition (active ingredients):

100 g contain:
0.525 g Didecyldimethyl ammonium chloride
0.120 g Polyhexanide

Listing:

- VAH-list
- ÖGHMP-list

Spectrum of efficacy:

- bactericidal
- yeasticidal
- limited virucidal*

Physical and chemical properties:

Appearance: clear, colourless liquid
Odour: neutral
pH-value: approx. 7 at 20 °C
Density: approx. 1 g/cm³ at 20 °C
Flash point: > 60 °C according to DIN 51755

ANTISEPTICA



ANTISEPTICA Dr. Hans-Joachim Molitor GmbH

Legal successor of ANTISEPTICA chem.-pharm. Produkte GmbH
D-50259 Pulheim, Carl-Friedrich-Gauß-Straße 7
Tel.: +49-2234-984 66-0, Fax: +49-2234-984 66-11
E-Mail: info@antiseptica.com, www.antiseptica.com

The company is certified according to **EN ISO 13485** and meets the Council's requirements of **Directive 93/42/EEC** on medical devices.

ANTISEPTICA chem.-pharm. Produkte GmbH

A-1020 Wien, Handelskai 388 / Top 641
Tel.: +43-1-374 66 00, Fax: +43-1-374 66 00-66
E-Mail: office@antiseptica.at, www.antiseptica.at

Antisepsis by Antiseptica – Antisepsis mit Vernunft

Application:

Surface disinfection with mechanical action; scrub-wipe disinfection:

Apply a sufficient amount (wet-wiping) of the undiluted, ready to use product to the surface by using a scrub-wipe disinfection with mechanical distribution. Ensure a thorough, complete moistening of the surface and allow to operate for the recommended contact time.

Massive contamination with organic material, e.g. blood, secretions, etc., should be removed before disinfection using a disposable cloth moistened with disinfectant, which should then be properly disposed.

For the use in combination with the wipes dispensing systems please refer to the recommendations on the product information leaflets.

Special advice:

- Wear adequate personal protective equipment when handling disinfectants.
- Avoid contact with eyes and mucous membranes.
- Clean and rinse surfaces, buckets and wiping utensils properly before first use or when switching products.
- Do not mix with other substances (e.g. anionic surfactants, aldehydes).
- Please follow local guidelines for the cleaning and disinfection of surfaces!
- Ensure adequate ventilation.

Forms of delivery:

- carton of 10 x 1 L plastic bottles
- carton of 2 x 5 L plastic canisters

also available:

- various application techniques that can be screwed onto bottles and canisters

The product is manufactured according to the international Quality Management System EN ISO 13485.

When handling the product, please note the information according to the Hazardous Substances Ordinance given in the safety data sheet.

Further information is provided by your expert advisor upon request or on our website.

Use disinfectants safely. Always read the label and product information before use.

Note for disposal:

The stock solution of Acrylan® can be disposed of with waste water, as it is common practice for cleaning agents. A neutralisation or inactivation before release into sewage is neither necessary nor ecologically worthwhile.

Surface disinfectant for medical devices according to Directive 93/42/EEC

CE 0482

Biocidal product according to BPR
BAuA Reg.-No.: N-21331

ANTISEPTICA



ANTISEPTICA Dr. Hans-Joachim Molitor GmbH
Legal successor of ANTISEPTICA chem.-pharm. Produkte GmbH
D-50259 Pulheim, Carl-Friedrich-Gauß-Straße 7
Tel.: +49-2234-984 66-0, Fax: +49-2234-984 66-11
E-Mail: info@antiseptica.com, www.antiseptica.com

The company is certified according to **EN ISO 13485**
and meets the Council's requirements of **Directive 93/42/EEC**
on medical devices.

ANTISEPTICA chem.-pharm. Produkte GmbH
A-1020 Wien, Handelskai 388 / Top 641
Tel.: +43-1-374 66 00, Fax: +43-1-374 66 00-66
E-Mail: office@antiseptica.at, www.antiseptica.at

Antisepsis by Antiseptica – Antisepsis mit Vernunft