



Alcoholic, readily soaked cleaning and disinfection tissues for the rapid disinfection of surfaces

Fields of application:

Descorapid® Tissues are readily soaked, alcoholic cleaning and disinfection tissues. They are perfectly suitable for a **rapid and easy disinfection** and cleaning of surfaces like for example operating tables, examination tables, stretchers, dressing carts, medicine cabinets, working surfaces, etc.; also in hospitals, doctors' practices and dentists' practices, as well as ambulances, critical care transport, or mobile care units.

Furthermore Descorapid® Tissues are suited for the cleaning and disinfection of all sorts of surfaces like for example tables and chairs, door handles and hand rails in the waiting areas of doctors' practices, in nursing homes, public institutions, or kindergarten, alcohol-resistant surfaces incl. PC keyboards in laboratory and diagnostics, in the office, at home or in transit.

Descorapid® Tissues are perfectly suited to be taken on car trips or on holidays.

Descorapid® Tissues are also suitable for the use in the cosmetic industry, cosmetic studios and beauty salons, as well as in the food industry and food-processing areas.

The product is free of aldehydes and QACs (quaternary ammonium compounds) as well as surfactants and colourants.

Additionally, Descorapid® Tissues are suitable for hygienic hand cleaning.

Recommendations for use:

Test (of disinfection solution) Concentration Contact time

VAH – Surface Disinfection with mechanical action | DGHM Standard Methods

VAH – Surface Disinfection: bactericidal, yeasticidal clean conditions	undiluted	5 min
--	-----------	-------

bactericidal, yeasticidal dirty conditions	undiluted	30 s
--	-----------	------

EN Methods – Surface Disinfection without mechanical action in food, industrial, domestic and institutional areas

EN 1276 (phase 2 / step 1) – bactericidal dirty conditions	undiluted	30 s
--	-----------	------

EN 1650 (phase 2 / step 1) – yeasticidal dirty conditions	undiluted	30 s
---	-----------	------

EN 13697 (phase 2 / step 2) – bactericidal, yeasticidal dirty conditions	undiluted	2 min
--	-----------	-------

DVV / RKI Guideline – virucidal efficacy

limited virucidal* (inactivation of enveloped viruses) (incl. HIV / HBV / HCV)	undiluted	30 s
--	-----------	------

Vacciniavirus	undiluted	30 s
---------------	-----------	------

BVDV (Bovine Viral Diarrhea Virus)	undiluted	30 s
------------------------------------	-----------	------

* according to RKI recommendation, Bundesgesundheitsblatt 01/2004

Special characteristics:

- easy, practical application
- flexible all-purpose disinfection
- free of aldehydes, QACs, surfactants, and colourants
- pleasant scent
- good cleaning properties

Spectrum of efficacy:

- bactericidal
- yeasticidal
- limited virucidal*

Listing:

- VAH-list (solution)
- IHO-list (food industry)
- ÖGHMP-list

ANTISEPTICA



ANTISEPTICA Dr. Hans-Joachim Molitor GmbH

Legal successor of ANTISEPTICA chem.-pharm. Produkte GmbH

D-50259 Pulheim, Carl-Friedrich-Gauß-Straße 7

Tel.: +49-2234-984 66-0, Fax: +49-2234-984 66-11

E-Mail: info@antiseptica.com, www.antiseptica.com

The company is certified according to **EN ISO 13485** and meets the Council's requirements of **Directive 93/42/EEC** on medical devices.

ANTISEPTICA chem.-pharm. Produkte GmbH

A-1020 Wien, Handelskai 388 / Top 641

Tel.: +43-1-374 66 00, Fax: +43-1-374 66 00-66

E-Mail: office@antiseptica.at, www.antiseptica.at

Application:

Take out one tissue at a time; afterwards always close lid again properly.

Surface disinfection with mechanical action; scrub-wipe disinfection:

Wipe (scrub-wipe disinfection) the entire surface with the ready to use disinfection tissue. Ensure a thoroughly, complete moistening of the surface; use further tissues if necessary (wet-wiping). Allow to operate for the recommended contact time.

Surface disinfection without mechanical action acc. to EN methods:

Wipe the entire surface with the ready to use disinfection tissue. Ensure a thoroughly, complete moistening of the surface; use further tissues if necessary (wet-wiping). Allow to operate for the recommended contact time.

All surfaces that come into contact with foodstuffs are recommended to be rinsed with clear water (of at least drinking water quality) after the recommended contact time has elapsed.

Massive contamination with organic material, e.g. blood, secretions, etc., should be removed before disinfection by using a Descorapid® Tissue or another disposable cloth moistened with disinfectant, which should then be properly disposed.

Handling of the box:

Open the package inside the box on one end. Put the package (formed to a roll) into the box. Pull the first tissue from the centre of the roll through the orifice in the lid of the box and put the lid back on.

Refilling of box

The lid including the inner orifice has to be disinfected properly before each refilling. Use an alcoholic surface disinfectant to thoroughly disinfect all lid surfaces on the inside and outside.

As soon as all surfaces have dried off you may proceed as described under "handling of the box". Please use only Descorapid® refill packs.

Disinfect your hands before each refill process and put on new protective gloves.

We recommend to completely refrain from the use of refill packs within high risk areas.

Special advice:

- Alcoholic disinfectants should not be used for the disinfection of large surfaces (danger of fire and explosion!). Output: max. 50 ml/m².
- Keep away from sources of ignition; no smoking. Hot surfaces must be cooled down before disinfection. Electric equipment must be switched to neutral/off and volt-free.
- Contains volatile substances. Ensure adequate ventilation.
- Not suitable for the disinfection of alcohol-sensitive surfaces (e. g. acrylic glass, alcohol-soluble lacquers).
- Avoid contact with eyes, mucous membranes and open wounds.
- Wear adequate personal protective equipment when handling disinfectants.
- Clean and rinse surfaces, buckets and wiping utensils properly before first use or when switching products.
- Please follow local guidelines for the cleaning and disinfection of surfaces!

Physical and chemical properties of the disinfection solution:

Appearance: clear, colourless liquid
Scent: alcoholic
pH-value: approx. 6 - 8 at 20 °C
Flash point: 21.5 °C according to DIN 51755

Content:

120 tissues soaked with 250 g disinfection solution; tissue size 14 cm x 22 cm

Composition (active ingredients):

100 g solution contain:
63.14 g Propan-2-ol

Forms of delivery:

- carton of 12 boxes of 120 tissues each
- 12 refill packs of 120 tissues

The product is manufactured according to the international Quality Management System EN ISO 13485.

When handling the product, please note the information according to the Hazardous Substances Ordinance given in the safety data sheet.

Further information is provided by your expert advisor upon request or on our website.

Use disinfectants safely. Always read the label and product information before use.

Biocidal product according to BPR
BAuA Reg.-No.: N-24416, N-55856

ANTISEPTICA



ANTISEPTICA Dr. Hans-Joachim Molitor GmbH
Legal successor of ANTISEPTICA chem.-pharm. Produkte GmbH
D-50259 Pulheim, Carl-Friedrich-Gauß-Straße 7
Tel.: +49-2234-984 66-0, Fax: +49-2234-984 66-11
E-Mail: info@antiseptica.com, www.antiseptica.com

The company is certified according to **EN ISO 13485**
and meets the Council's requirements of **Directive 93/42/EEC**
on medical devices.

ANTISEPTICA chem.-pharm. Produkte GmbH
A-1020 Wien, Handelskai 388 / Top 641
Tel.: +43-1-374 66 00, Fax: +43-1-374 66 00-66
E-Mail: office@antiseptica.at, www.antiseptica.at

Antisepsis by Antiseptica – Antisepsis mit Vernunft